

WarmAir® FilteredFlo®

Convective Warming System

Safety



Outcomes

Selection

WarmAir® FilteredFlo Convective Warming System

Improve patient safety and patient outcomes...

The WarmAir system from CSZ provides safe, effective air warming therapy to your patient.

Improve Surgical Outcomes with Patient Temperature Management

Approximately 60-70% of surgical patients become hypothermic. Recent studies have proven that warming patients before, during and after surgery, and other specialty procedures can markedly improve clinical outcomes. Keeping the patient's temperature near normal during colorectal surgery has been shown to dramatically reduce wound infections, speed healing and shorten hospital stays by 2.5 days.¹

Convective air warming provides the following patient benefits:

- Reduces hypothermia during most surgical procedures
- Works to maintain patient core temperature keeping patients normothermic
- Promotes healing and reduces wound infections
- Warms cold patients post-operatively
- Helps improve patient recovery time, minimizing the length of hospital stays

¹Sessler DI, Kurz A, Lenhardt R. "Perioperative Normothermia to Reduce the Incidence of Surgical Wound Infection and Shorten Hospitalization." *NE Journal of Medicine*.1996; 334 (19): 1209-1215.

WarmAir® Warming Unit

- Reduced air circulation, lower risk of infection
- 100% of air is HEPA filtered
- Quiet operation
- Four levels of therapy
- Compact and lightweight
- High and low temperature safeties
- Meets UL 2601 standards required in 2010



WarmAir® Specifications

Dimensions:	8.75" w X 8.75" d X 13.5" h (22.2cm X 22.2cm X 34.3cm)
Weight:	13.5 lbs. (6.1kg)
Temperature Settings:	Fan Only / Ambient Low: 90°F (32.2°C) Medium: 100°F (37.8°C) High: 110°F (43.3°C)
Heater:	1200 watt
Airflow:	35 cfm
Filtration:	0.2 micron

FilteredFlo Warming Blankets

FilteredFlo single-use convective air blankets offer unmatched performance and value. There are a wide range of models available to choose from to fit both the needs of your patient and your budget.

✓ Effective Patient Warming

All CSZ warming blankets provide uniform warm air distribution and retain heat for effective patient warming. FilteredFlo blankets have built-in flaps that tuck around the patient, providing superior therapy and eliminating the need for additional cover sheets or blankets.

✓ Patient Safety

CSZ's WarmAir warming system provides double air filtration to protect your patient from airborne bacterial contaminants. This is accomplished by heated air that is first filtered through a HEPA filter on the warming unit then filtered a second time through CSZ's unique FilteredFlo warming blankets.

✓ Patient & Caregiver Comfort

Conforming to the patient's body, FilteredFlo blankets deliver evenly distributed warm air without a hint of blowing air. Warm air diffuses through the lower drape material without causing discomfort to the patient or medical personnel.

✓ Easy to Access

FilteredFlo blankets are easy to manage, will not "fly away," and may be folded back for easy access to your patient.

✓ Economical

Since additional sheets and blankets are not needed with FilteredFlo blankets, laundry and processing costs are also reduced.

FilteredFlo Upper Body Warming Blanket



The upper body blanket is typically used for abdominal or lower body procedures warming the upper torso and arms.

- Attached head drape
- Two inlet ports
- Attached securing ties
- Easy peel one-piece tape



Model 443



Model 442

FilteredFlo Lower Body Warming Blanket



The lower body blanket is utilized during surgical procedures that require access to the upper half of the body.

- Non-heated foot drape
- Extra width to fit all size patients
- Tuckable flaps

FilteredFlo Adult Warming Blanket



The adult blanket provides full coverage for patients of all sizes, is generously sized and includes extra material for the shoulders and feet. Used for pre-operative, post-operative and outpatient care.

- No top sheet or blanket required
- Easy patient access
- Low profile



Model 243



FilteredFlo Torso Warming Blanket



Used during surgical procedures that require access to the lower half of the body. The torso blanket includes a clear drape to cover and warm the intubated patient's head.

- Two inlet ports
- Attached head drape
- Easy access to IV on left or right side



Model 344

FilteredFlo® Sterile Cardiac Warming Blanket



The sterile cardiac blanket is utilized during open heart procedures.

- Actively heats groin area
- Easy access through perforated opening
- Oversized to fit all patients



Model 542



Model 244

FilteredFlo Pediatric Warming Blanket



Provides full coverage for pediatric patient care. Blankets include extra material for the shoulders and feet along with a foot drape to reduce the risk of thermal injury. Pediatric blankets are used for pre-operative, intra-operative, and post-operative care. Ideal for use in the PACU, ICU and OR.

- No top sheet or blanket is required
- Low profile
- Easy patient access

FilteredFlo® Neonate/Pediatric Warming Blanket



The neonate/pediatric blanket provides warmth to neonates, newborns and small children. The blanket can be placed over or under the patient and with a clear drape that can be placed on top of the patient to help retain heat. Neonate/pediatric blankets are used for pre-operative, intra-operative, and post-operative care. Ideal for use in the PACU, ICU and OR.

- Drainage holes included
- Pre-attached tape to secure blanket



Model 241




How do CSZ FilteredFlo Warming Blankets Compare to Others?

CSZ is able to deliver WarmAir Therapy to your patients utilizing the patented non-woven delivery surface of the FilteredFlo blanket. Our FilteredFlo blankets require no perforation thereby avoiding unwanted and potentially dangerous particles in your operating room.

Questions to Consider

- Are you concerned with patient safety?
- Do you currently use similiar blankets in your O.R.?
- Would you use these blankets on your patient?

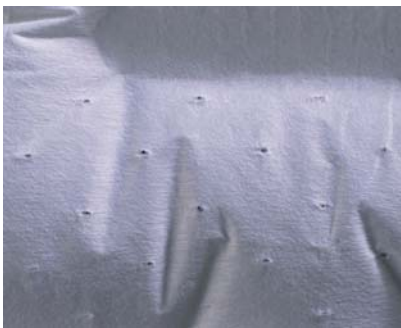

Competitive blanket 1




Competitive blanket 2



Competitive blanket 3 (Note visible particles)



Competitive blanket particles

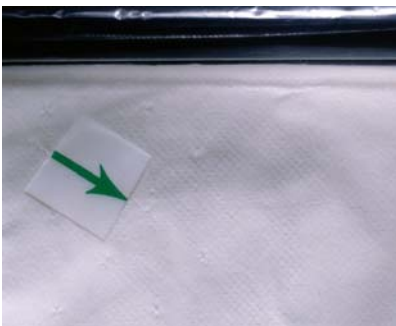

Un-opened competitive blanket

Which warming blanket would you like to use on your patients?



CSZ's FilteredFlo blanket

The Difference in CSZ Patient Warming Systems

The WarmAir Model 135 is compact (13.5"/34.3cm tall), lightweight (13.5 pounds/6.1 kg) and attaches easily to operating room tables, stretchers, bed ends, side rails or IV poles. Optional Model MST-135 Mobile Stand offers added convenience.

CSZ uses Filtered Air Warming as opposed to Forced Air Warming

The unique patented design of FilteredFlo blankets permits use of a lower velocity blower to supply gently moving, clean air. The filtered air warming method minimizes air currents that may spread contaminants to your patient.











Filtered Air method with CSZ's FilteredFlo blankets



Forced Air method with competitive brand

FilteredFlo® Blanket Specifications


FilteredFlo® Warming Blankets				
TYPE	SIZE	METALLIC	NON-METALLIC	
Adult	92in x 50in (233.7cm x 152.4cm)	Model 143	Model 243	
Pediatric	56in x 40in (142.2cm x 101.6cm)	Model 144	Model 244	
Neonate	29in x 30in (73.6cm x 76.2cm)	---	Model 241	
Upper Body	88in x 30in (223.5cm x 76.2cm)	Model 343	Model 443	
Lower Body	58in x 40in (147.3cm x 101.6cm)	Model 342	Model 442	
Torso	43in x 40in (109.2cm x 101.6cm)	---	Model 344	
Sterile Cardiac	72in x 50in (182.8cm x 127cm)	---	Model 542	
Tube	28in x 66in (71.1cm x 167.6cm)	---	Model 145	

WarmAir FilteredFlo blankets are also available with a metallic top layer.




Warming Tube™

The warming tube is ideal for use in the PACU, ICU, Cath. Lab and Radiology. Since patients are enclosed in warmth through “surround-air”, distortion of radiographic images or interference with the critical performance of imaging equipment can be avoided.

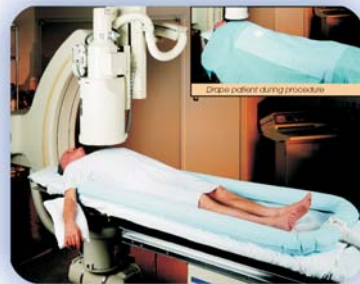

Authorized European Representative
 CEpartner4U BV
 Esdoornlaan 13
 3951 DB Maarn
 The Netherlands
 Tel: +31 (0) 6-516-536.26

CSZ
 Cincinnati Sub-Zero
 Medical Division

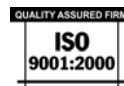

12011 Mosteller Road
 Cincinnati, Ohio 45241-1528
 513-772-8810 800-989-7373
 Fax: 513-772-9119
 www.cszmedical.com
 ©Cincinnati Sub-Zero Products, Inc., 2004
 99197 9/04